




Safety Introducer with Arrow® Peelable Sheath

FASTER. SAFER. SIMPLER.

Reduce vessel cannulation-associated risks with the PICC WAND, the newest Vascular Access innovation offered by Teleflex Medical using Accelerated Seldinger Technique.

The PICC WAND combines a 21 ga. echogenic needle, nitinol wire and Arrow peelable sheath over dilator into an all-in-one safety device that reduces exposure to blood during insertion and eliminates the need to reach for separate components, improving sterile technique.

The PICC WAND is designed to improve vascular access by reducing the risk of vessel trauma, loss of cannulation, needle-stick injury and embolism associated with Modified Seldinger Technique.

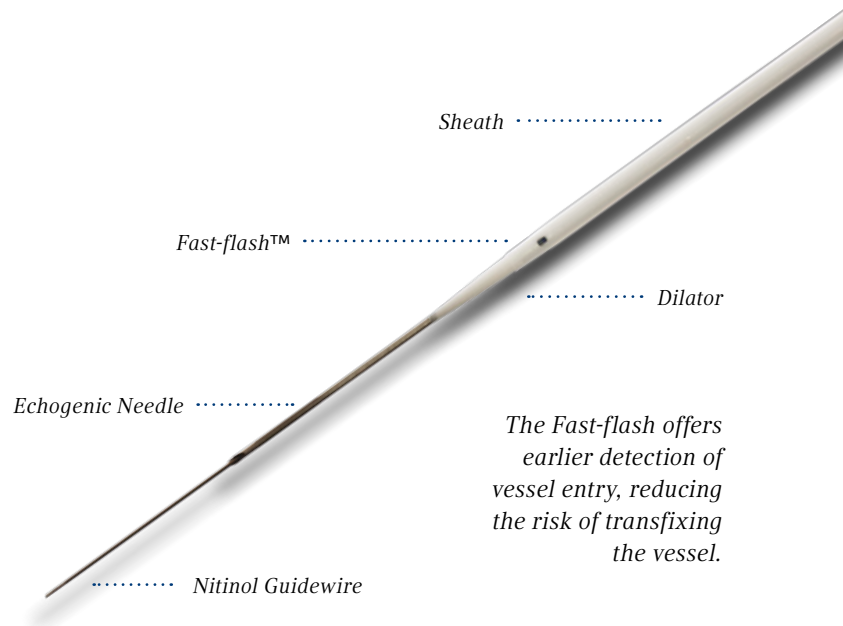

Sheath

Fast-flash™

..... Dilator

Echogenic Needle

The Fast-flash offers earlier detection of vessel entry, reducing the risk of transfixing the vessel.

..... Nitinol Guidewire

THE PICC WAND


with Arrow Peelable Sheath



The unique design of the PICC WAND features a ZeroEdge™ transition between needle, dilator and sheath.

FOUR BASIC STEPS* OF ACCELERATED SELDINGER TECHNIQUE

For the PICC WAND with Arrow Peelable Sheath

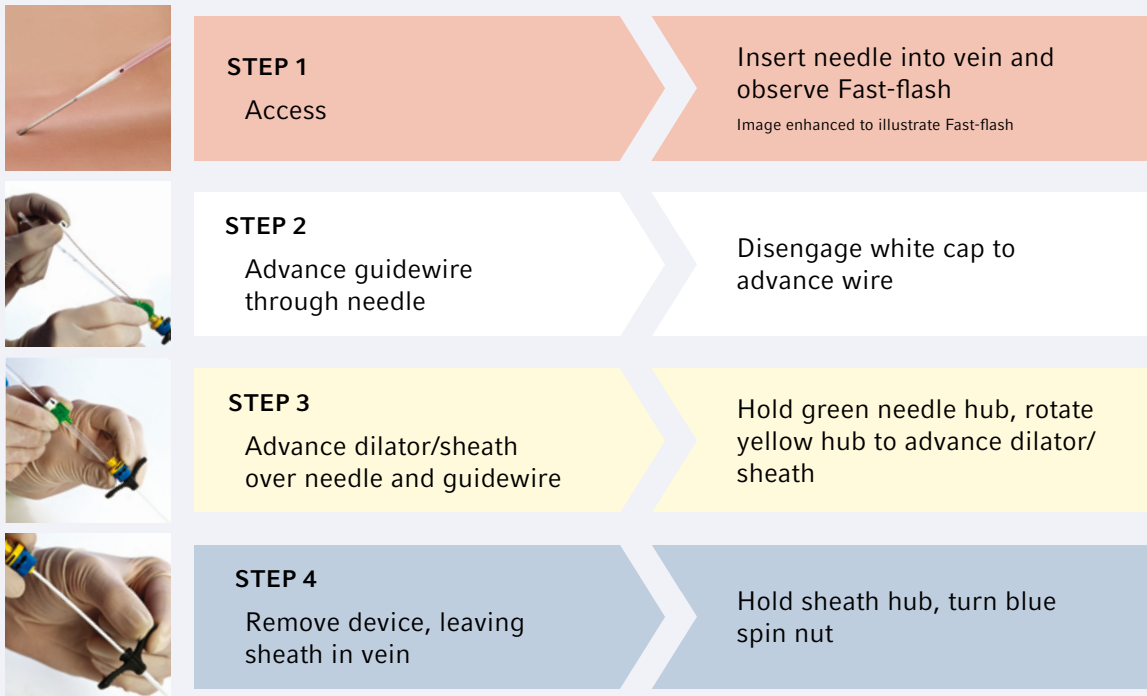

ORDERING INFORMATION

Item #	Description	ARROW Hub Color
PI-00940-AST	4 French PICC WAND Safety Introducer with Arrow Peelable Sheath. Accepts 4 Fr. PICC or midline catheters up to 1.4 mm max OD	Red
PI-00945-AST	4.5 French PICC WAND Safety Introducer with Arrow Peelable Sheath. Accepts 4.5 Fr. PICC or midline catheters up to 1.5 mm max OD	Yellow
PI-00950-AST	5 French PICC WAND Safety Introducer with Arrow Peelable Sheath. Accepts 5 Fr. PICC or midline catheters up to 1.7 mm max OD	Grey
PI-00955-AST	5.5 French PICC WAND Safety Introducer with Arrow Peelable Sheath. Accepts 5.5 Fr. PICC or midline catheters up to 1.8 mm max OD	Dark Blue
PI-00960-AST	6 French PICC WAND Safety Introducer with Arrow Peelable Sheath. Accepts 6 Fr. PICC or midline catheters up to 2.0 mm max OD	Dark Green

* Prior to use of any medical device, please read all package inserts, warnings, precautions and instructions. Failure to do so may result in severe patient injury or death.

Fast-flash, the PICC Wand and ZeroEdge are trademarks or registered trademarks of Access Scientific Inc.

Arrow is a registered trademark of Teleflex Incorporated or its affiliates.

©2010 Teleflex Incorporated. Lit. No. 2009-0329 Rev. 00 Tel: 866.246.6990 Int: 919.361.3914 www.teleflexmedical.com

Teleflex
MEDICAL